

Disposal Guidelines for Prevention™ HLD8 High Level Disinfectant



This document has been developed in accordance with current applicable waste disposal regulations. It is intended for use as a guideline on best disposal practices for Virox Technologies Inc. products (5 Gallons or less) only. At no time should this document replace existing documents established by the facility, municipal, state or federal bodies.

The overall goal of this guideline is to dispose of the solution in a manner that would abide by municipal disposal regulations and occupational health and safety regulations.

PROTECTIVE BARRIERS

1. Disposable gloves. Gloves should be changed as required, i.e., when torn, or when hands become wet inside the glove.
2. Protective eye wear (goggles, face shield or mask with eye protection) should be worn.

2% PREVENTION™ HLD8

For small quantities (< 1 Gallon):

1. Turn on the cold water tap
2. Pour the solution down the drain with water running
3. Allow water to run for 3 minutes
4. Triple rinse empty container
5. Dispose empty container into recycling

Alternatively, for facilities with access to pH indicator/pH measuring unit:

1. Adjust the pH to between 6 and 8 using common baking soda (approximately 12 grams or 1 tablespoons per litre)
2. Pour the solution down the drain
3. Allow tap water to run for 1 minute
4. Triple rinse empty container
5. Dispose empty container into recycling

REFERENCES

1. Environmental Protection Act EPA (P.R.O. 1990, Regulation 347, General-Waste management)
2. Workplace Hazardous Materials Information System (WHMIS)
3. Hazardous Products Act (Controlled Products Regulations)
4. Hazardous Products Act (Chapter H-3)
5. A Resource Guide to Pollution Prevention in the Health Care Sector - 2001